

Installing the Oasys Software

Step 1 : Download the install program from the website

Go to the Oasys download page

(http://www.occupationalasthma.com/occupational_asthma_pageview.aspx?id=4443)

Click on the relevant link, probably “Oasys”, but maybe “Oasys MWD” or “Oasys Asma-1” if you want to use Oasys with the Mini Wright Digital meter or the Vitalograph Asma-1 meter.



Download Oasys (free)

There are three versions of Oasys at the moment, a hand entry version as well as versions that download data automatically from the Asma-1 and Mini Wright Digital peak flow meters. They are free.


Download and run the appropriate setup program from the list below to install Oasys. You will need to [register with the website](#) in order to use it (this is also free).

- [Oasys](#) (Hand Entry)
- [Oasys MWD](#) (For Clement Clarke Mini Wright Digital)
- [Oasys Asma-1](#) (For Vitalograph Asma-1)

Further Information

The latest version (April 2012) has added the [Firewall / Connection Checker](#), which can diagnose problems with connecting to the Oasys server and the Opening Records window now default

Then click on the installer to download it to your computer.

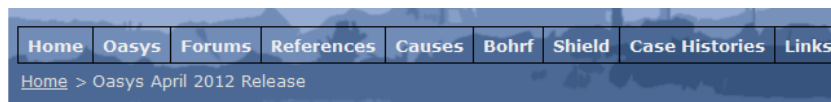

Oasys April 2012 Release


Click on the link below to download this release of Oasys. This will download the installation program to your computer. You should run this file to install Oasys.

You will need to [register with the website](#) in order to use the software (which is free). All personally identifiable details in the records you save will be kept on your computer. All anonymous details will be saved on the Oasys Server. See our [data protection section](#) for more details. No personal details are ever sent to or stored on the Oasys Server.

[resources/resource144_install_oasys_april_2012.exe](#)

Step 2: Run the install program


Double click on the installer. If a dialog box like this one appears click “Yes”.

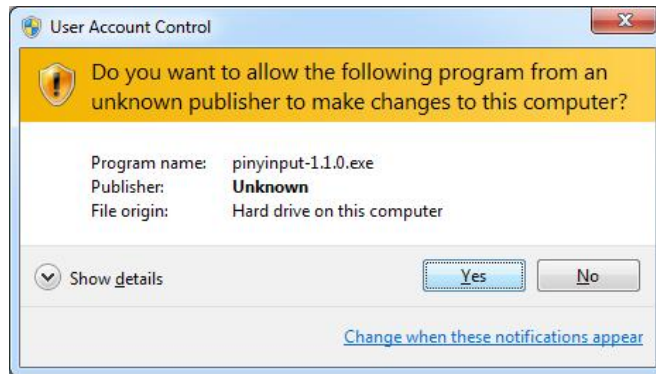

The welcome dialog will appear first, click “Continue”



Choose where you would like Oasys to install.

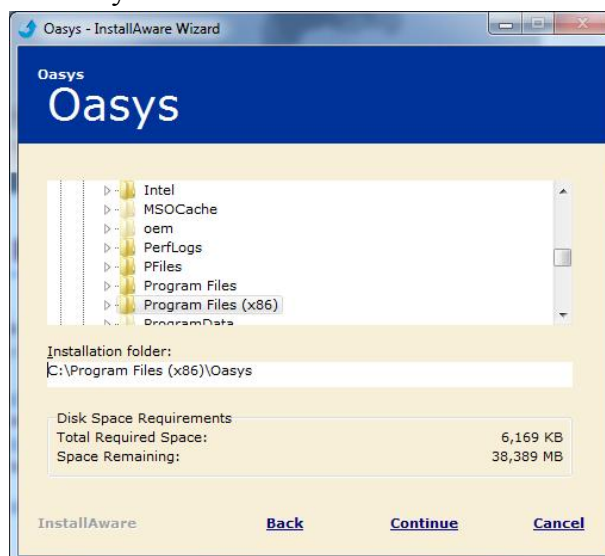

Choose the start menu folder and decide whether to install it for all users on the computer or just for you.



The installer will now start copying files.



Step 3: Start using Oasys!

Oasys will now be available on the start menu, as an icon on the desktop.